

MOTOR STEP 

MOTORSTEP

For Greater Mobility and
Improved Wellbeing



CONTACT

01296 709630
info@motorstep.net
www.motorstep.net



GIVE BACK CONTROL

Effortlessly lift inside or outside
your home

Enable people with disabilities to live happily in their own home for longer. Simply and safely improve outcomes with Motorstep - the cost effective way to help people to manage steps without bulky ramps or expensive building work.

Mobile options available to enable holidays and boost wellbeing

- ▶ Designed for people with disabilities
- ▶ Free standing - place next to the existing step, no bulky ramps needed!
- ▶ Weather proof
- ▶ Discretely folds up for storage: set up in seconds
- ▶ Easy touch control buttons & strong support frame to make lifting easy and safe
- ▶ Mains or battery powered options



MOTORSTEP

The ideal equipment to promote **self management and improve wellbeing**. Enable patients to participate in daily life within their own homes.

Facilitate holidays and wellbeing breaks. Ideal for use in caravans and motorhomes.

Allow **everyone** to live comfortably in their **own home**

MOTORSTEP IN ACTION!

CASE STUDY

HOW MOTORSTEP HELPED ONE CUSTOMER RETAIN HER FREEDOM

A deteriorating health condition meant that our customer needed to plan for her future as her mobility was worsening. It was time to plan for the future and it was important that this lady remained in her own home.

The problem

Our customer had investigated ways to gain access to her property.
Option 1: A wheelchair lift but this could cost £13,000 to install
Option 2: A ramp. To install a ramp would cost £9,000 and it would drastically change the appearance of her property.

The solution

A local contractor removed the steps at the front of her property and installed a concrete base for the Motorstep to sit on. The total cost for site preparations and the supply of a Motorstep was just over £2300 - works were completed in under 2 weeks.

The result

Motorstep provides an extremely cost effective - and unobtrusive - way for the customer to safely remain in her own home and enjoy quality of life

A cost effective and safe way for the owner to enter and exit her own home

£2300 overall cost - 75% cheaper than alternative solution

fast installation in less than 2 weeks

100% safe property access and unobtrusive appearance



CASE STUDY

MOVING AROUND SAFELY INSIDE A HOME THANKS TO MOTORSTEP

Two steps covering a height difference of 18 inches between the living area and the kitchen were a barrier for this customer due to her age and failing mobility.

The problem

This lady could no longer be independent as she could not navigate steps. She was reliant on the local authority and her family for food and hot drinks.

The local authority recommended a wheelchair lift and after means testing, determined a cost to our customer of £4000. A six month wait, the cost, and the amount of space the lift would take up in her small kitchen mean that this was not a viable option.

The solution

Working with her OT, a local carpenter fitted a box over a section of the two steps to create a single step. Motorstep was delivered and plugged in - and our customer effortlessly and safely could now access her kitchen.

The result

Our customer was able to stay in her own home and enjoy much greater independence. This improved her wellbeing and reduced her dependence on the LA and her family.

A cost effective solution allowing our customer to keep living in her own home.

retained independent living in own home

fast installation compared to a 6 months wait

more cost effective than the alternative solution

WHICH MOTORSTEP IS PERFECT FOR YOUR CLIENT?

Motorstep is available in a range of models for inside and outside use, with different weight capacities and lift heights. Motorstep may be powered by mains power or battery.

Visit our website www.motorstep.net to view all options.



Product pictured: Motorstep MS 600H



SEE MOTORSTEP IN ACTION ON OUR WEBSITE OR YOUTUBE CHANNEL. JUST SEARCH 'MOTORSTEP'!

HOW TO USE YOUR MOTORSTEP



STEP 1

Once your Motorstep is in place, simply step onto the platform that is just 1.5 inches/38mm off the ground.



STEP 2

Hold onto the two handles and press the green button.



STEP 3

The Motorstep platform will gently rise in tandem with the central moving column you are holding, providing fantastic reassuring support.



STEP 4

Once you reach your required height, release the green button and the Motorstep will gently stop.



STEP 5

Simply step off the platform into your home, onto your decking or out into the garden.



STEP 6

Leave your Motorstep in position ready for your return.

ABOUT MOTORSTEP



Proudly designed
and made in
Great Britain



MOTORSTEP is a family business based in Buckinghamshire, England.

We established Motorstep in 2013 after personal experience meant we saw firsthand just some of the restrictions faced by those with disabilities. We realised that a mobile, motorised, step-on lift would return independence and encourage an active lifestyle. Motorstep was born!

Today, there are multiple Motorsteps that suit different situations and needs. Each one is designed to help negotiate a step with ease and confidence.

Customer satisfaction is our highest priority in order for us to establish ourselves as a leader in the mobility aid market for many years to come. We work with all our customers to assess their mobility and individual needs to ensure Motorstep is right for them before a purchase is made.

We ship to customers in Great Britain and Europe direct from our UK factory. We also have a distribution network worldwide across USA, New Zealand, and Australia, visit www.motorstep.net for details.

How to contact and order from motorstep

If you have any questions please do not hesitate to contact us. Our dedicated family team is always here to help.

Want to try the Motorstep?

We have a demonstration area at our Buckinghamshire factory where patients can try out the Motorstep. In some cases, we can offer a demonstration in a patient's home - contact us for details. We can work with Local Authorities, NHS providers, Charities and the end user to provide Motorstep to patients.

We can take orders over the phone and delivery is normally within 7-10 working days.



CONTACT

01296 709630
info@motorstep.net
www.motorstep.net

MOTORSTEP

Unit S2, Westcott Venture Park
Westcott, Buckinghamshire
HP18 0XB, England



@motorstep