MOTORSTEP

Greater mobility around the home





CONTACT

01296 709630 info@motorstep.net www.motorstep.net





TAKE BACK CONTROL

Effortlessly lift yourself inside or outside your home

Motorstep helps you to enjoy freedom and independence. It allows you to safely move from your home out into your garden, up a step into your front door, negotiate an internal step, or access high beds and furniture.

Perfect for inside and outside brick, mobile and static homes

- ▶ Designed for people with disabilities
- ► No costly installation or building work needed
- Free standing place next to the existing step, no bulky ramps needed!
- Weather proof
- Discretely folds up for storage: set up in seconds
- ► Easy touch control button & strong support frame to make lifting easy and safe
- Mains or battery powered options



MOTORSTEP

A simple, effective way to **overcome a step at home in a controlled and easy way**. Eliminating the need for bulky ramps or expensive building work,

Motorstep is a cost effective way to access areas of your home that have become a problem.

Live comfortably in your **own home**

HOW TO USE YOUR MOTORSTEP



My Husband has been struggling for years with the steps into our static home and it got to the point where he no longer wanted to go out when we visited on holiday. We looked into having a ramp fitted but the cost was so expensive, My husband finds Motorstep so easy to use. I would recommend a Motorstep to anyone who has a problem with the step into the home

MRS GILROY, CHICHESTER





Once your Motorstep is in place, simply step onto the platform that is just 1.5 inches/38mm off the ground.

STEP 2



Hold onto the two handles and press the green button.

STEP 3



The Motorstep platform will gently rise in tandem with the central moving column you are holding, providing fantastic reassuring support.

STEP 4



Once you reach your required height, release the green button and the Motorstep will gently stop.

STEP 5



Simply step off the platform into your home, onto your decking or out into the garden.

STEP 6



Leave your Motorstep in position ready for your return.



SEE MOTORSTEP IN ACTION ON OUR WEBSITE OR YOUTUBE CHANNEL.
JUST SEARCH 'MOTORSTEP'!

WHICH MOTORSTEP IS PERFECT FOR YOU?

Motorstep is available in a range of models with differing lift heights depending on the intended use plus 2 power options. Take a look at our handy table to help you choose the best option to meet your needs.



Create an infill on your decking to provide a stable platform for when you step off your Motorstep.

Contact us for details.







MS600H

FOR ALL YOUR STANDARD HOME STEPS

The leading motorised step for indoor and outdoor use at home.

Love your independence! Motorstep lifts you effortlessly up and down a step allowing you the freedom to go wherever you choose. Pick MS600H for all your standard home steps.

MS600H	
For homes, static homes and mobile homes	
Step on height	1.5 inches / 38mm
Lift rise (height)	24 inches / 600mm
Maximum weight	20st / 125kg / 280lb
Max Width	455mm
Depth (Folded)	200mm
Depth (Platform Lowered)	650mm
Overall Height	1200mm
Platform Size	350x450mm
Powered by	230 volt / 12 volt power supply from any standard 13 amp power outlet. Lithium ion battery powered option also available

GOOD TO KNOW

- Motorstep lifts can be purchased VAT free in the UK by registered disabled customers or customers with chronic illnesses
- Motostep is fitted with wheels to help you easily roll it into place



MS800H

FOR HIGHER STEPS AROUND THE HOME

The leading motorised step for indoor and outdoor use at home.

Love your independence! Motorstep lifts you effortlessly up and down a step allowing you the freedom to go wherever you choose.

Enjoy even greater independence and freedom with the MS800H.

MS800H	
For homes, static homes and mobile homes	
Step on height	1.5 inches / 38mm
Lift rise (height)	32 inches / 800mm
Maximum weight	20st / 125kg / 280lb
Max Width	455mm
Depth (Folded)	200mm
Depth (Platform Lowered)	650mm
Overall Height	1350mm
Platform Size	350x450mm
Powered by	230 volt / 12 volt power supply from any standard 13 amp power outlet. Lithium ion battery powered option also available

- ▶ The security eye bolt prevents theft and the removable master key stops unwanted use
- ▶ Motorstep is fully weatherproof and can be left outside in any weather
- Fitted safety edges automatically stop descent if an object is detected under the platform



Motorstep has been such a great help in my home, I have a large step between my kitchen and living room and being an old house there was not enough room for a ramp. Motorstep is so easy to use and it's allowed me to stay in my home without costly building work.

MRS HARRIS. WINCHESTER

POWER AND OPTIONAL EXTRAS

POWER

Motorstep MS600H and MS800H are both supplied with a 230v (UK and Europe) or 110v (North America) power supply fitted with country specific plug. Simply plug your Motorstep into any standard power outlet.

A 12v extension cable up to 8 metres long can be supplied if your Motorstep will be positioned away from a convenient socket.

Don't want to use a power supply? Try the Li-ion battery! Contact us for more information.



ACCESSORIES

Easy Lift Wheels

Both the MS600 and MS800 can be fitted with wheels to provide an alternative method of lifting for storage.



Support Strap

Both the MS600 and MS800 Motorstep step lifts can be fitted with a support strap that easily attaches to the central moving column. It is designed to support the user as they are being lifted.





Wander lead/Remote

Both the MS600 and the MS800 Motorstep can be fitted with external control buttons that allows the Motorstep to be operated by a second person if the main user requires a little extra assistance.



Even though the Motorstep is weather proof and designed to be used outside we can supply a bespoke cover. An eyelet allows the use of a security chain. Also suitable for Freedom models.





CASE STUDY

MOVING FROM GROUND LEVEL UP ONTO DECKING

A cost effective and safe way for the owner to enter and exit her own home



How Motorstep helped one customer retain her freedom

A deteriorating health condition meant that our customer needed to plan for her future. Her mobility was worsening and navigating steps was becoming increasingly difficult. It was time to plan for the future and it was important that this lady remained in her own home.

£2300 overall cost - **75**% **cheaper** than alternative solution

fast
installation in
less than
2 weeks

100% safe property access and unobtrusive appearance

The problem

When our customer approached Motorstep, she had already investigated ways to gain access to her property.

Option 1: A wheelchair lift but this could cost £13,000 to install

Option 2: A ramp. To install a ramp would cost £9,000 and it would drastically change the appearance of her property.

The solution

We established with the customer that the steps at the front of her property could be removed and a gate could be installed in the railings to allow access to the Motorstep. A concrete base for the Motorstep to sit on was included in the site preparation. The customer employed a local contractor to carry out the required alterations.

The total cost for supply of the Motorstep and the site preparations was just over £2300, which was considerably cheaper than the option of a wheelchair lift installation or a ramp installation. The works were completed in under two weeks.

The results

Motorstep provided an extremely cost effective - and unobtrusive - way for the customer to safely enter and exit her static home. This reassured our customer as she knew that if her condition deteriorated, she could still access and continue living in her own home.

The cost saving of the Motorstep compared to the alternatives was very welcome to a retired person on a fixed annual income. Needless to say, our customer was very pleased that she discovered Motorstep!







ABOUT MOTORSTEP





Proudly designed and made in Great Britain

MOTORSTEP is a family business based in Buckinghamshire, England.

We established Motorstep in 2013 after personal experience meant we saw firsthand just some of the restrictions faced by those with disabilities. We realised that a mobile, motorised, step-on lift would return independence and encourage an active lifestyle. Motorstep was born!

Today, there are multiple Motorsteps that suit different situations and needs. Each one is designed to help negotiate a step with ease and confidence.

Customer satisfaction is our highest priority in order for us to establish ourselves as a leader in the mobility aid market for many years to come. We work with all our customers to assess their mobility and individual needs to ensure Motorstep is right for them before a purchase is made.

We ship to customers in Great Britain and Europe direct from our UK factory. We also have a distribution network worldwide across USA, New Zealand, and Australia, visit www.motorstep.net for details.

HOW TO CONTACT AND ORDER **FROM MOTORSTEP**

If you have any questions please do not hesitate to contact us. Our dedicated family team is always here to help.

Want to try before you buy?

We have a demonstration area at our Buckinghamshire factory. Visit us and try out the Motorstep for yourself. In some cases, we can offer a demonstration in your own home - contact us for details.

We can take orders over the phone and delivery is normally within 7-10 working days.



CONTACT

01296 709630 info@motorstep.net www.motorstep.net

MOTORSTEP

Unit S2, Westcott Venture Park Westcott, Buckinghamshire HP18 0XB, England





